

REDGUM BRAND
Daily Living and
Lifestyle Products

KNEE WALKER

RG40KWHD



USER MANUAL

Please read and keep these instructions

redgumbrand.com.au



REDGUM Brand owns the copyright to the Knee Walker user manual.
Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

REDGUM Brand strives to provide the highest quality products possible.
Products are subject to change without notice.

© Copyright **REDGUM Brand** 2021



REDGUM Brand is part of **FOR-DE Group Pty Ltd**

redgumbrand.com.au

CONTENTS

1. **INTRODUCTION** - INTENDED USE
2. **PRODUCT OVERVIEW** - FEATURES / SPECIFICATIONS
3. **ASSEMBLY** - UNPACK and INSPECT SUPPLIED PARTS
4. **ASSEMBLY** - HANDLEBAR
5. **ASSEMBLY** - HANDLEBAR (continued)
6. **ASSEMBLY** - KNEEPAD
7. **ASSEMBLY** - WIRE BASKET
8. **USAGE GUIDELINES** - DO's + DON'Ts
9. **PRODUCT CARE** - INSPECTIONS / CLEANING
10. **WARRANTY**
11. **REDGUM BRAND** PRODUCT RANGE
12. **NOTES**



INTRODUCTION

Thank you for purchasing this quality **REDGUM Brand** product.

If you have any issues or questions about the **Knee Walker** please contact your supplier or REDGUM Brand direct:

T: +61 8 9248 4180

F: +61 8 9248 9452

E: sales@for-de.com.au

A: 1 Business Way, Malaga Western Australia 6090

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

Only use your **Knee Walker** for the purpose for which it is intended, which is as a transportation solution that can adjust to suit the users individual requirements.

IMPORTANT NOTE

This equipment **MUST** be assembled by a competent person.

Maximum User Weight: 180kg
Exceeding the weight limit could cause damage to the product and could put the user at risk.



1.

redgumbrand.com.au

PRODUCT OVERVIEW

FEATURES

- Easy to assemble
- Adjustable height padded contoured kneepad
- Height adjustable handlebars
- Folds down for transport & storage
- Anti slide hand grips
- Large wheels for excellent mobility & stability
- Strong lightweight construction
- Removable wire basket



SPECIFICATIONS (All dimensions shown are nominal)

Overall Dimensions: 1050mm (L) x 500mm (W) x 900mm (H) - at lowest setting

Kneepad Size: 380mm (L) x 190mm (W)

Kneepad Height: 530 - 640mm (from floor to top of seat)

Handlebar Width: 490mm

Handlebar Height: 900 - 1050mm (from floor to top of handlebar)

Wheel Diameter: 200mm / 8"

Product Weight: 15.5kg

Maximum User Weight: 180kg

Please contact **REDGUM Brand** if you would like any further information on your **Knee Walker**.

ASSEMBLY - UNPACK and INSPECT SUPPLIED PARTS

Ensure you have been supplied with the following parts:

PARTS:



IMPORTANT NOTE

Check the product for any visible damage. If you see any damage or suspect a fault, **DO NOT USE YOUR PRODUCT** and contact your supplier for support.

3.

redgumbrand.com.au

ASSEMBLY - HANDLEBAR

The handlebar requires location into the frame:

1. Ensure the front of the handlebar is facing toward front of knee walker.
2. Line up and insert the handle screw into hole.
3. Screw on washer and nut to lock handlebar in position. (Ensure it is fixed in position before going to the next step)



IMPORTANT NOTE:

Ensure it is fixed in position before going to the next step.




4.

redgumbrand.com.au

ASSEMBLY- HANDLEBAR (continued)

4. Rotate the quick release lever up and over onto the top mounting plate.
5. Pivot the lever over to lock the steering column in position.
6. To adjust the handle height remove the tri wheel bolt, move the tube up and down to the desired height setting, align holes and using the bolt fix it into position.

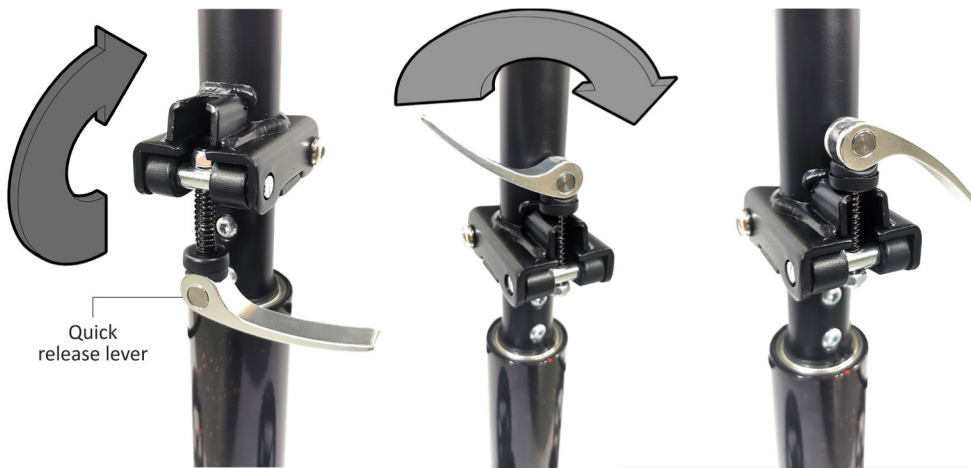

IMPORTANT NOTE:
Pivot the lever over to lock the steering column in position.



IMPORTANT NOTE:
To adjust height of Handlebar:
- Undo bolt
- Adjust height
- Align holes
- Reinstall knob



Tri wheel bolt

5.

ASSEMBLY - KNEEPAD

The kneepad requires location into the rear part of the frame:

1. Insert tube on kneepad into upright on frame.
2. Line up holes at required height setting and insert screw into hole.
3. Screw on knob to lock kneepad in position.



Lever screw

IMPORTANT NOTE:

- To adjust height of Kneepad:
- Undo knob
 - Adjust height
 - Align holes
 - Reinstall knob

IMPORTANT NOTE:

Make sure the kneepad knob is screwed in position before you continue with assembly.



6.

ASSEMBLY - WIRE BASKET

The Knee Walker comes with a wire basket.
To fit the wire basket to the handlebar:

1. Align the holes on basket mount plate with the hangers on the handlebar.
2. Attach the basket to the hangers as shown.



IMPORTANT NOTE:

Do not overload the wire basket as it may affect steering

USAGE GUIDELINES - DO'S + DON'TS

Please follow these guidelines to get the optimal benefit from your Knee Walker.

Failure to follow these guidelines may result in damage to the Knee Walker and put the user at risk.

180kg Maximum User Weight.

Ensure ALL FOUR WHEELS remain on ground at all times when the Knee Walker is being used.

Do not lean too far to the side or backward causing the wheels to leave the ground.

The Knee Walker should be used on level ground (Avoid uneven ground where possible).

Use the handbrakes to regulate speed where possible - especially when going downhill.

Do not corner at high speed as the Knee Walker may topple over.

Do not overload the wire basket as it may affect steering.
(Maximum load: 2kg)

IMPORTANT NOTE:

Always set up the Knee Walker to suit the user. Adjust handlebar and knee pad to a comfortable and safe working height as required.



PRODUCT CARE - INSPECTIONS / CLEANING

INSPECTIONS

We suggest that you check your Knee Walker visually periodically for signs of damage or wear prior to use. Overall check that no components are loose or worn.



Withdraw the Knee Walker immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

IMPORTANT NOTE

Do not fit any parts or accessories to your Knee Walker that are manufactured by another company. To do so will void your warranty and may seriously compromise your safety.

CLEANING

Clean your Knee Walker using a non-abrasive cleaner or mild detergent with a soft cloth.

Abrasive cleaners or abrasive cleaning pads could damage the Knee Walker and should not be used.



WARRANTY

Your REDGUM Brand **RG40KWHD - Knee Walker** is covered by a **12 Month Warranty** from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

If you have any issues or questions about your Knee Walker please contact your supplier or REDGUM Brand direct on:

T: +61 8 9248 4180
F: +61 8 9248 9452
E: sales@for-de.com.au
A: 1 Business Way, Malaga
Western Australia 6090



**Please keep this User Manual
in a safe place for future reference.**

REDGUM Brand - Product Range

REDGUM Brand offer a comprehensive range of products including:

- Seat Walkers**
- Side Folding Seat Walkers**
- Walking Frames**
- Wheelchairs**
- Walking Sticks + Quads**
- Crutches**
- Reachers**
- Shower / Bathroom products**
- Toilet products**
- Cushions: Air / Foam**
- Electric Lift Chairs**
- Electric Beds**
- Portable / Fixed Aluminium Ramps**
- Exercisers**
- Personal Alarms**



REDGUM Brand stock a large range of Spare Parts for all REDGUM brand products.

Please contact REDGUM Brand for further information:


T: +61 8 9248 4180

F: +61 8 9248 9452

E: sales@for-de.com.au

A: 1 Business Way, Malaga Western Australia 6090

W: www.redgumbrand.com.au

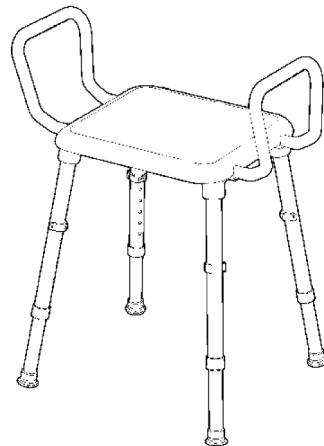

NOTES:

DISTRIBUTOR:

REDGUM BRAND
Daily Living and
Lifestyle Products



Proudly Australian Owned and Operated

redgumbrand.com.au