


TARANTO

FABRIC 4 MOTOR LIFT CHAIR

LC0401-2

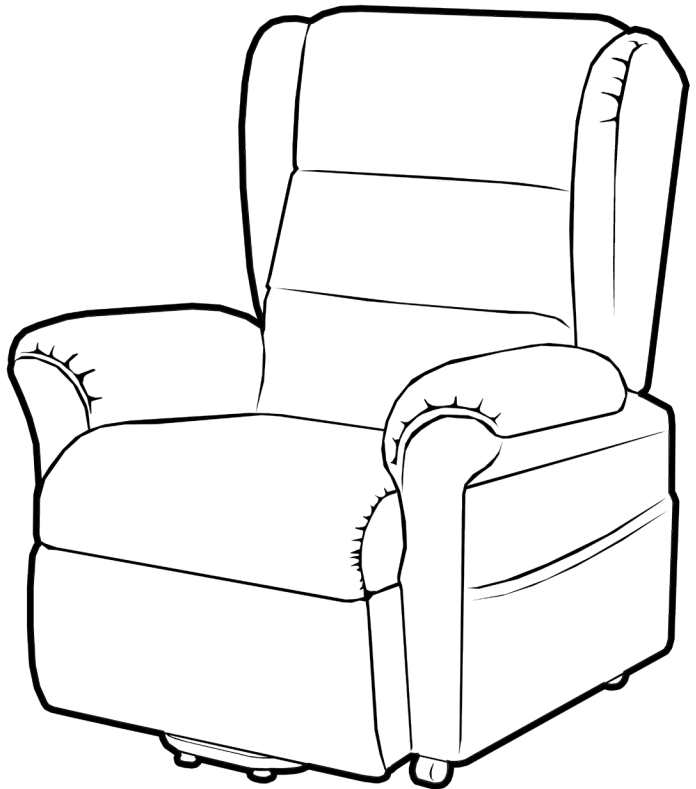

USER MANUAL

EDITION 1

Please read and keep these instructions



REDGUM Brand owns the **copyright** to the TARANTO LIFT CHAIR user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

REDGUM Brand strives to provide the highest quality products possible. Products are subject to change without notice.


© Copyright **REDGUM Brand** 2021



REDGUM Brand is part of **FOR-DE Group Pty Ltd**

CONTENTS

1. **INTRODUCTION** - INTENDED USE
2. **PRODUCT OVERVIEW** - FEATURES / SPECIFICATIONS
3. **ASSEMBLY** - UNPACK and INSPECT SUPPLIED PARTS
4. **ASSEMBLY** - ATTACH BACKREST TO CHAIR BASE
5. **ASSEMBLY** - ATTACH WINGS TO BACKREST
6. **ASSEMBLY** - ATTACH CASTORS
7. **ASSEMBLY** - HANDSET CONNECTION
8. **ASSEMBLY** - CABLE CONNECTION
9. **ASSEMBLY** - BATTERY BACKUP
10. **ASSEMBLY** - TRANSFORMER CONNECTION
11. **USAGE GUIDELINES** - CHAIR POSITIONING
12. **USAGE GUIDELINES** - HANDSET CONTROLS
13. **USAGE GUIDELINES** - DO's + DON'Ts
14. **PRODUCT CARE** - INSPECTIONS / CLEANING
15. **WARRANTY**
16. **REDGUM BRAND** PRODUCT RANGE

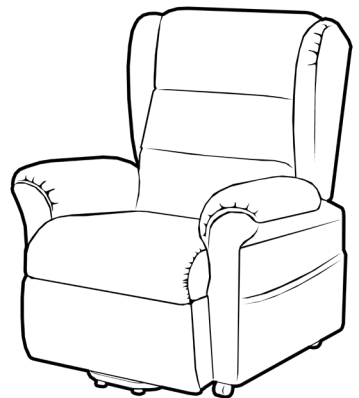

INTRODUCTION


Thank you for purchasing this quality **REDGUM Brand** product.

If you have any issues or questions about the **TARANTO Lift Chair** please contact your local distributor or REDGUM Brand direct:

T: +61 8 9248 4180

E: sales@for-de.com.au

A: 1 Business Way, Malaga Western Australia 6090

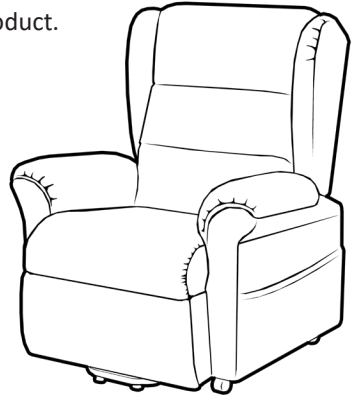

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

Only use your Taranto Lift Chair for the purpose for which it is intended, which is as a single seating armchair that can lift, rest and recline.

Read and follow all instructions, warnings and notes in this manual and all other accompanying literature before attempting to operate this product for the first time. In addition, your safety depends upon you, as well as your dealer, carer or healthcare professional in using good judgement.

IMPORTANT NOTE


This equipment **MUST** be assembled by a competent person.

It is recommended that a second person is used to assist with connecting the backrest onto the chair.

PRODUCT OVERVIEW

FEATURES

Lift / Rest / Recline armchair
Seat raise and tilt function
Backrest recline function
Legrest elevation function
Lumbar adjustment function
Headrest angle function
Comfortable and stylish
Available in a range of colours (refer to product sheet)
Fabric Upholstery
Quality German Okin electric actuators
Battery back up
Ergonomic handset with large backlit buttons
Strong lightweight construction

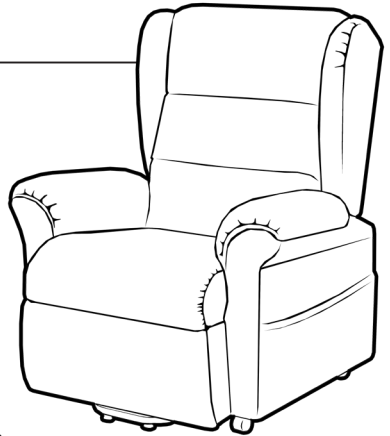

SPECIFICATIONS

(All specifications may vary (+/- 10mm) as each chair is handcrafted)

Overall (Sitting): **905mm (W) - 940mm (D) - 1080mm (H)**

Seat: **470mm (W) - 500mm (D) - 500mm (H) - 93° (Angle)**

Armrest: **210mm (W) - 500mm (L) - 170mm (H)**

Backrest: **560mm (Between wings) - 670mm (H) - 93 - 180° (Angle range)**

Legrest: **620mm (W) - 430mm (L) - 90 - 180° (Angle range)**

Lumbar: **450mm (W) - 220mm (H) - approx. 75mm (Adjustment depth range)**

Headrest: **580mm (W) - 260mm (H) - 0 - 45° (Angle range)**

Required distance from wall: **650mm**

Maximum User Weight: **160kg**


Product Weight: Base - **50kg** / Backrest - **15kg** / Combined - **65kg**

Please contact **REDGUM Brand** if you would like any further information.

ASSEMBLY- UNPACK and INSPECT SUPPLIED PARTS

Ensure you have been supplied with the following parts:

PARTS:

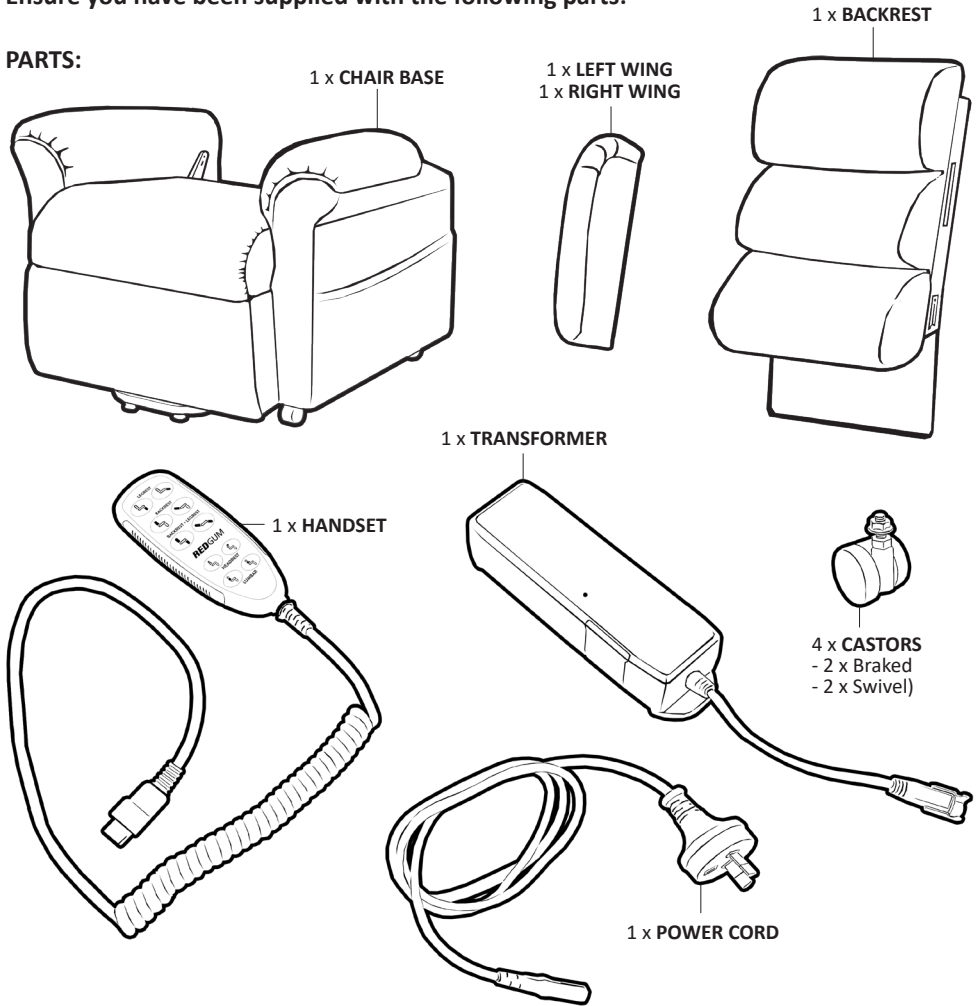

TOOLS REQUIRED:

(Not Supplied)

2 x 13mm FLAT SPANNERS

BATTERIES REQUIRED:

(Not Supplied)

2 x 9 Volt



IMPORTANT NOTE

Check the product for any visible damage. If you see any damage or suspect a fault, **DO NOT USE YOUR PRODUCT** and contact your supplier for support.


ASSEMBLY- ATTACH BACKREST TO CHAIR BASE

Please follow these steps to attach the backrest to the chair base:

1. Position the chair base as shown
2. Align the backrest over the chair base with the padded front facing toward front of chair base.
3. Fit the backrest to the chair base by sliding the mounting brackets located on either side of the backrest onto the mounting plates located inside the arm rests at the rear of the chair base.
4. Push down on the backrest until you hear two loud 'Clicks' indicating that **BOTH** connection points are locked in position.
5. Pull up on the backrest to check that it is securely locking down in place.

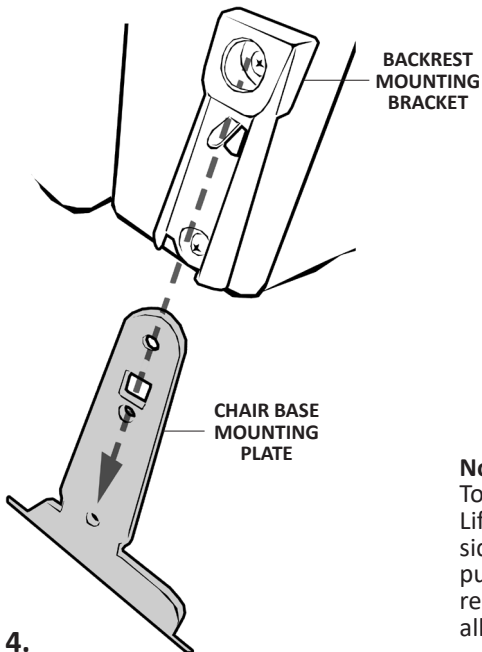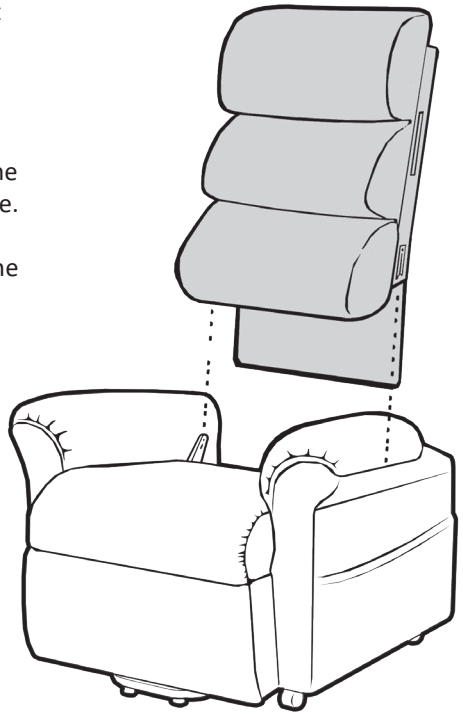




Note:
To remove the backrest:
Lift the triggers on either side of the backrest and pull up on backrest to release the lock and allow for removal.

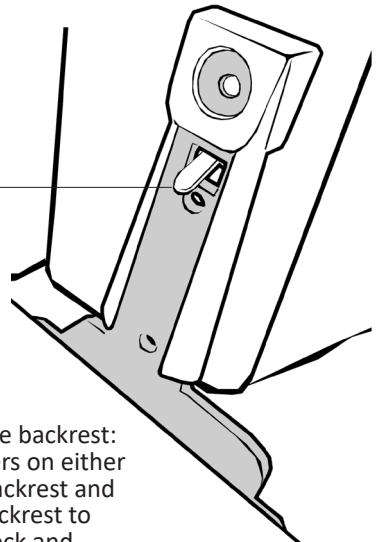

ASSEMBLY- ATTACH WINGS TO BACKREST

Please follow these steps to attach the wings to the backrest:

1. Align the bracket on the wing with the bracket on the side of the backrest.
2. Slide the wing down onto the bracket.
3. Push wing all the way down until fully located onto side of backrest.
4. Repeat process for other wing.

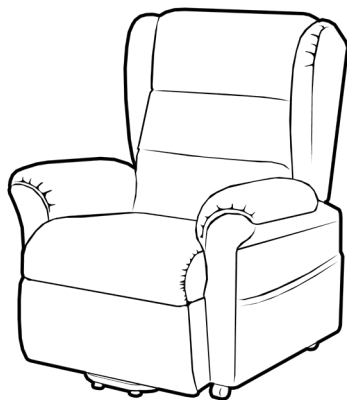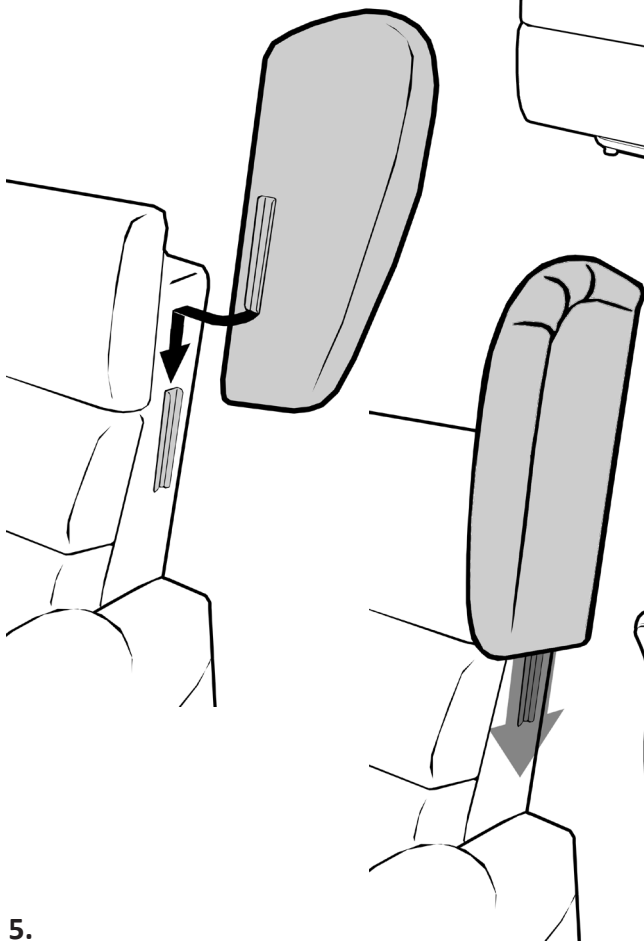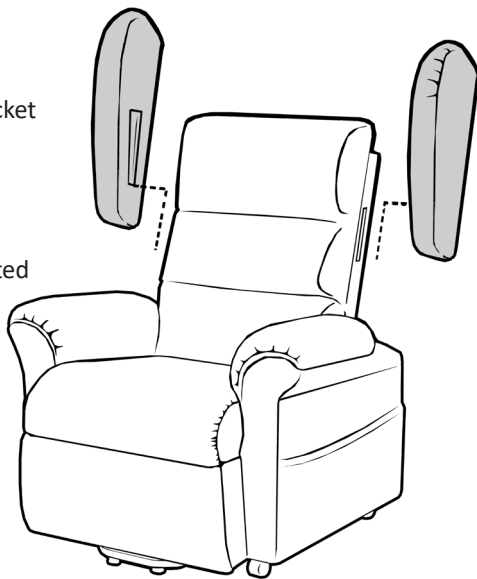



5.

ASSEMBLY - ATTACH CASTORS

The Taranto Lift Chair comes with castors that can be fitted to the chair base if required. The lift chair needs to be flipped onto its front as shown to gain access to the underside of base frame.




Bolt the relevant castor to the mount brackets on the base frame as shown below.

- 2 x **SWIVEL** castors at the **FRONT** of the chair
- 2 x **BRAKED** castors at the **REAR** of the chair

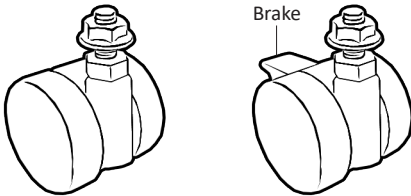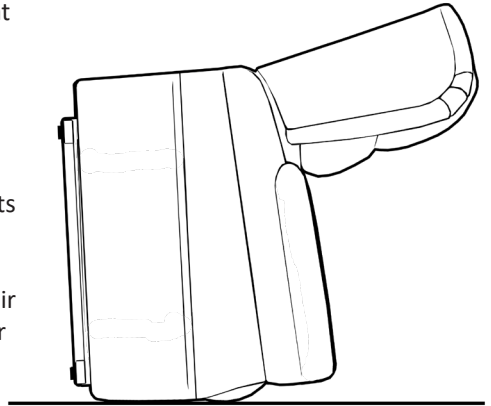



NOTE:

Use 2 x 13mm flat spanners to hold lower nut in position and tighten upper nut.

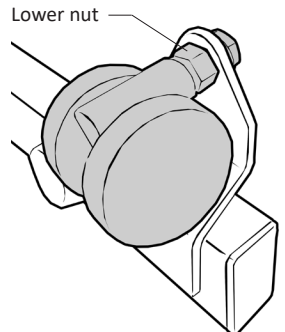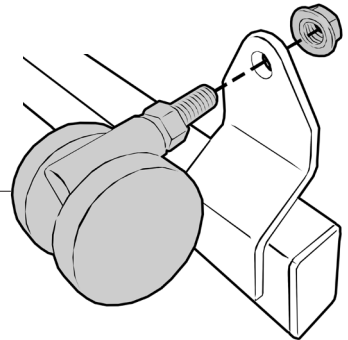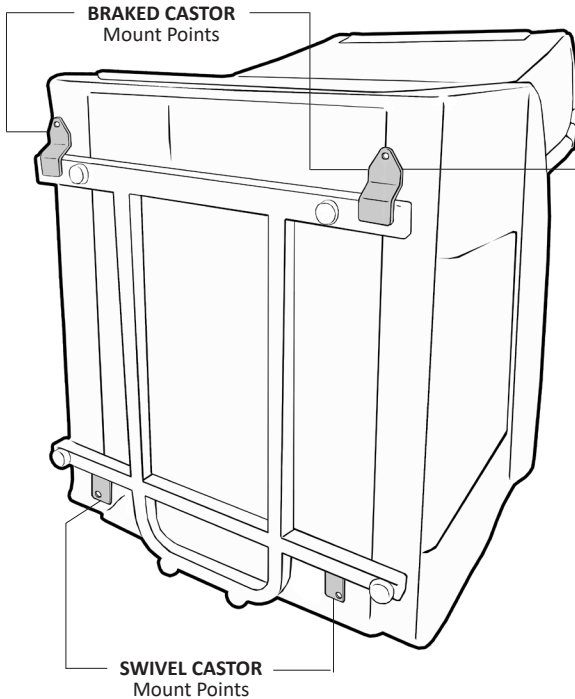

ASSEMBLY- HANDSET CONNECTION

The Taranto lift chair comes with a handset that needs to be connected to the lift chair PRIOR to being plugged into mains power.

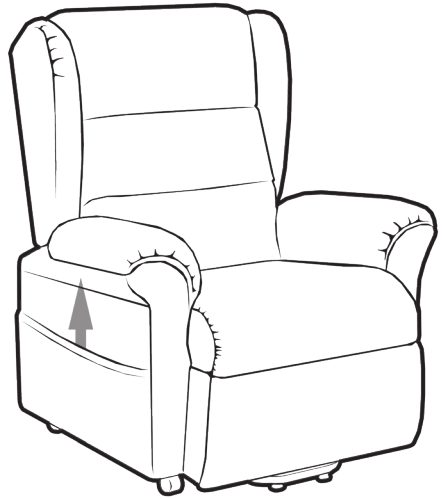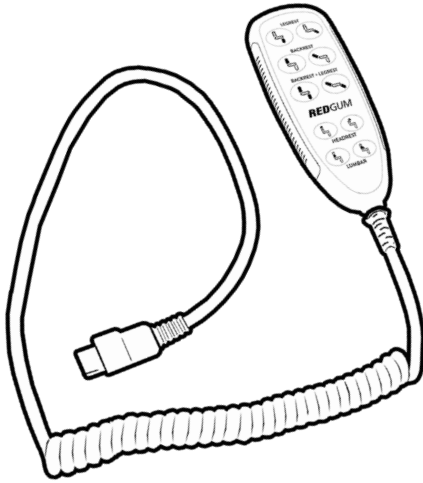

Flip the lift chair back onto the castors.

Reach into the pocket located on the side of the chair and pull out the cable.


Align the pins on the plug of the handset with the corresponding holes in the cable end (Flat surfaces on plugs to be inline).

Firmly push cables together.

Rotate the cable lock clip up until you hear a loud 'Click' indicating that cable ends are now locked together.

Important Note:



Always store the handset and cable in the storage pocket when not in use to avoid any accidental damage.

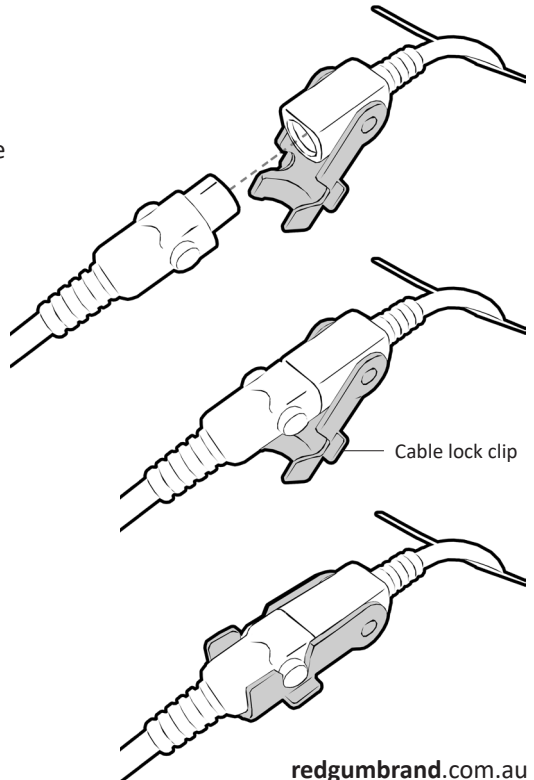

ASSEMBLY - CABLE CONNECTION

The Taranto lift chair has two cables located at the rear of the chair **behind the cover** that require connection prior to the chair being plugged into mains power.

1. Locate the two cables as shown.
2. Match cables with markings together.
3. Align the pins with the corresponding holes in the cable end.
4. Firmly push cables together.
5. Rotate the cable lock clip up until you hear a loud 'Click' which indicates that cable ends are now locked together.

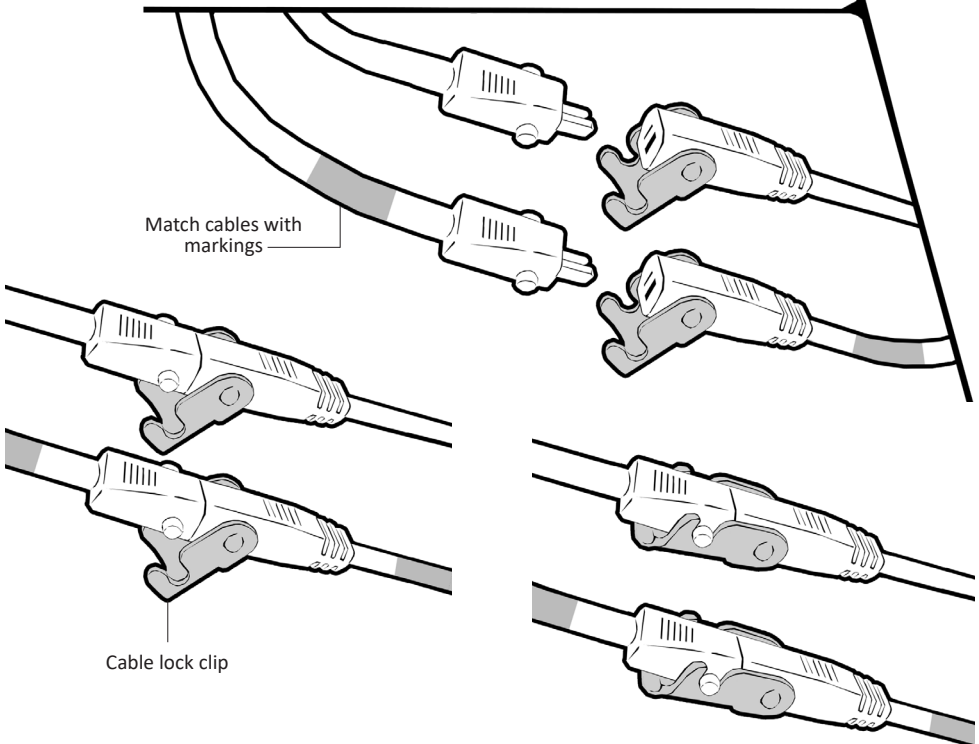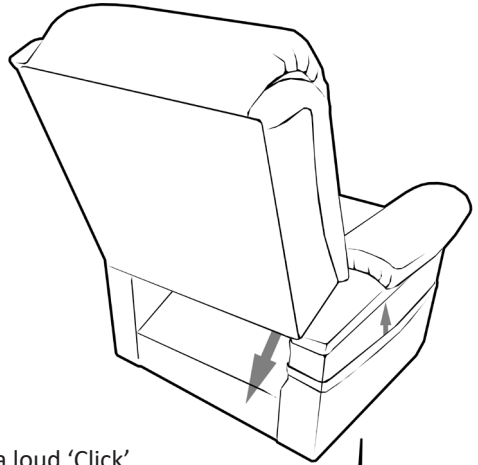


ASSEMBLY - BATTERY BACKUP

The Taranto Lift Chair is fitted with battery backup as standard. This allows for the lift / rest / recline / headrest / lumbar functions to still operate if the mains power is disrupted. It is recommended that main power is always used to operate the lift chair and the battery backup is only used to return the chair to a sitting position so the occupant can remove themselves from the lift chair.

To enable the battery backup please follow these steps:

1. Ensure the mains power is **NOT** plugged in.
2. Slide the cover on the transformer up to access the battery compartment.
3. Attach 2 x 9V batteries (NOT included) to the connection wiring.
4. Fit batteries inside cavity and refit cover.


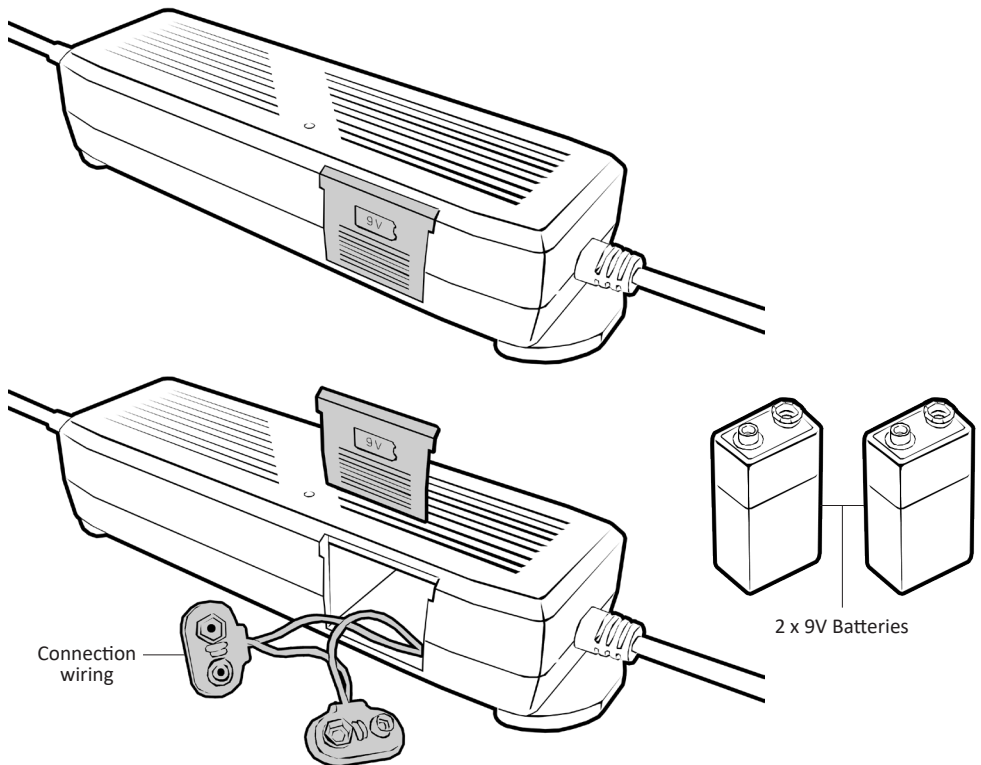


Important Note:




Remove the batteries if the chair is not plugged into mains power and in use. The lift chair will draw power from the batteries even when not in use.

ASSEMBLY- TRANSFORMER CONNECTION

The Taranto Lift Chair comes with a transformer that needs to be connected to the lift chair.

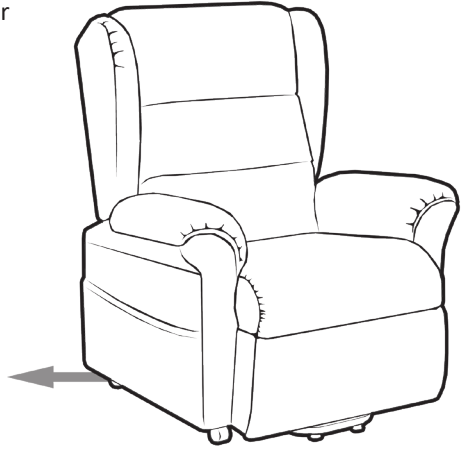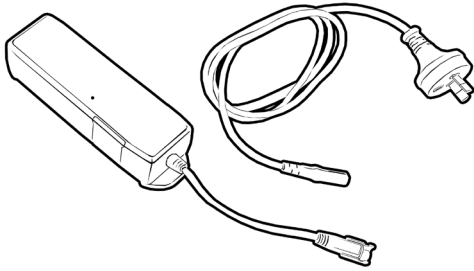

1. Locate the cable found at the rear of the lift chair on the underside.
2. Align the pins on the plug from the chair with the corresponding holes in the cable end (Flat surfaces on plugs to be inline).
3. Firmly push cables together and rotate the cable lock clip up until you hear a loud 'Click' indicating that cable ends are now locked together.
4. Plug transformer cable into transformer as shown below.
5. Plug cable into mains power (240V).

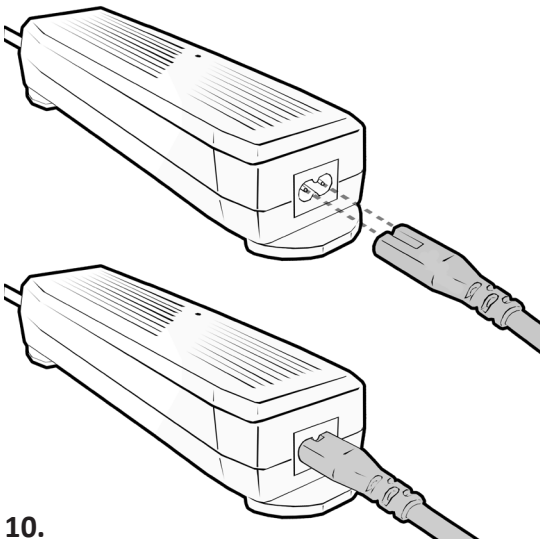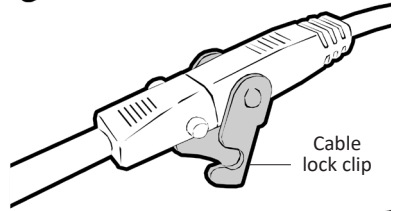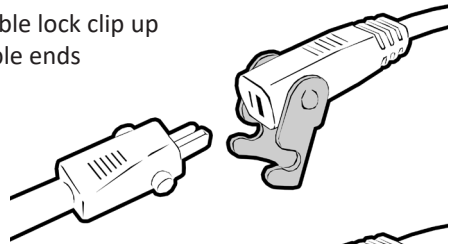



Important Note:

Always keep cables clear of base frame to avoid any potential damage.

USAGE GUIDELINES - CHAIR POSITIONING

Prior to using the Taranto Lift Chair please follow these steps to set up the chair in a safe manner:

- Position the chair on a flat and level floor.
- The lift chair should be placed near a power point. (We recommend that the chair be plugged directly into the power source where possible)
- Allow sufficient room in front and behind the chair for reclining backrest and extending legrest:
 - **650mm** from wall
 - **500mm + in front**

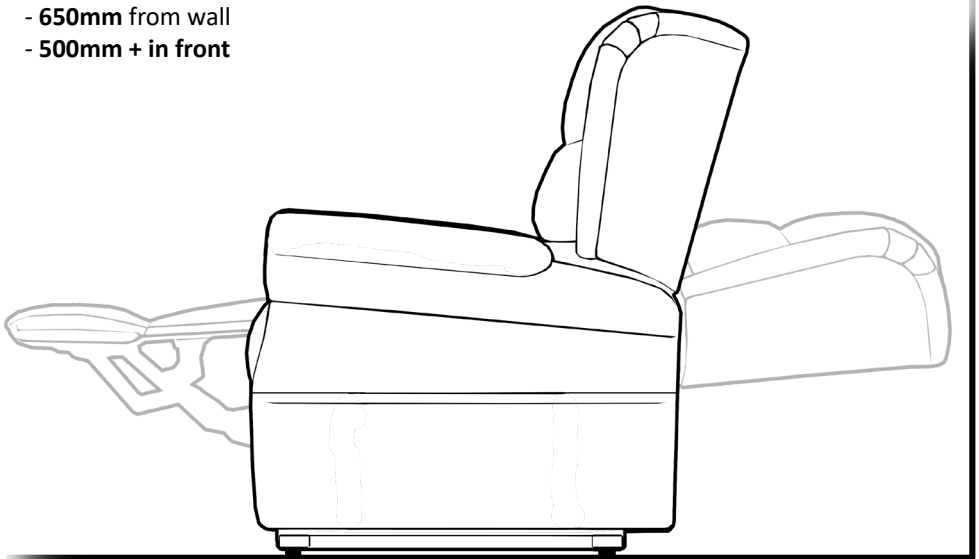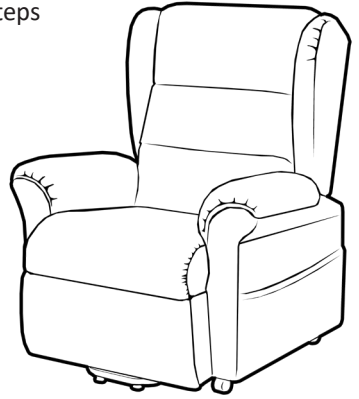


- Try to maintain at least 1 metre between lift chair and other furniture.
- Avoid placing the chair in the direct sun (Material colour will fade over time with sunlight).
- DO NOT use lift chair outdoors (Indoor use only).
- Plug lift chair into mains power - LED indicator on transformer will show green.

USAGE GUIDELINES - HANDSET CONTROLS

The Taranto Lift Chair comes with a backlit handset that controls the movement of the backrest, legrest, headrest, lumbar and overall chair position.

Depress the relevant button to achieve the required movement:


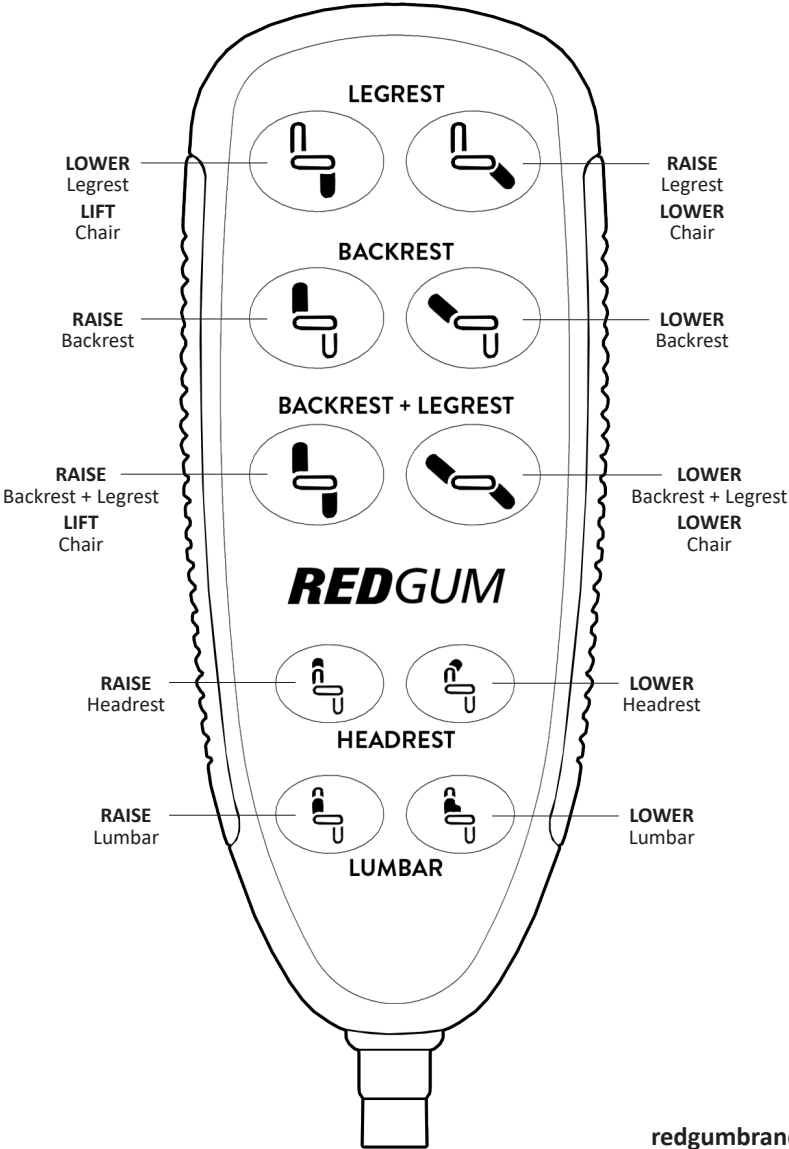


USAGE GUIDELINES - DO'S + DON'TS

Please follow these guidelines to get the optimal benefit from your Taranto Lift Chair.

Failure to follow these guidelines may result in damage to the lift chair and put the user at risk:

Maximum User Weight is **160kg**.
Do not overload lift chair!

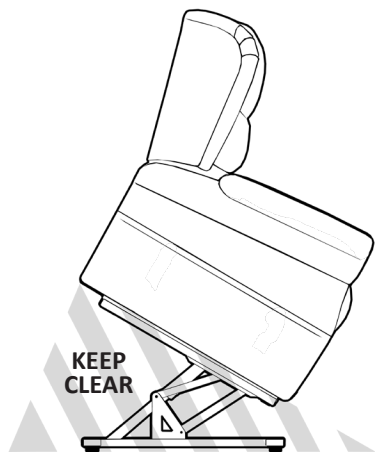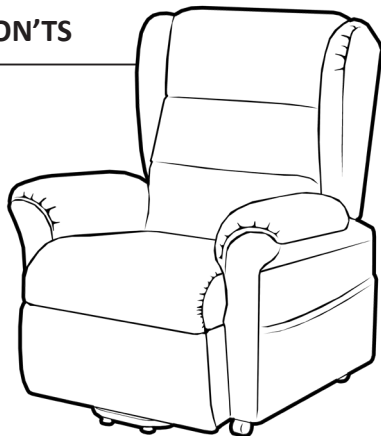

IMPORTANT NOTE:
PINCH POINTS EXIST ON LIFT CHAIRS
Keep clear of the lift and recline mechanisms at all times !

- Ensure areas behind and in front of the lift chair are clear at all times.
- Adjust the base frame feet if the lift chair is not level on the ground.
- Only the occupant or a qualified carer should operate the lift chair.
- Always leave the lift chair in the upright position when not in use.
- Ensure that only ONE person uses the lift chair at a time.
- Do not allow anyone to sit on the legrest when raised or on backrest when reclined.
- Always store the handset in the side pocket when not in use.
- Do not expose the lift chair to excessive heat, humidity or dust.
- When moving the lift chair hold from the base - not backrest or armrests.

If you are unsure about any of these precautions please contact your local distributor or **REDGUM Brand** direct for further information.

PRODUCT CARE - INSPECTIONS / CLEANING

INSPECTIONS

We suggest that you visually check your Taranto Lift Chair periodically for signs of damage or wear prior to use. Pay particular attention to the following areas:

- Cabling
- Handsets
- Transformer and battery backup
- Actuators (motors)
- Coverings (look for pulls / tears / gaps / stitch fraying)

Overall check that no components are loose or worn.



Stop using lift chair immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your local distributor or REDGUM Brand direct for advice and service support.

CLEANING

Clean your lift chair coverings using a non-abrasive cleaner or mild detergent with a soft cloth.


Spot clean any spills as soon as possible to reduce the amount of ingress into fabric seams.

Abrasive cleaners or abrasive cleaning pads could damage the lift chair coverings and should not be used.

DO NOT expose any of the electrics to any moisture:

- Actuators
- Handsets
- Cabling.

Keep these components dry at all times and away from any temperature extremes.

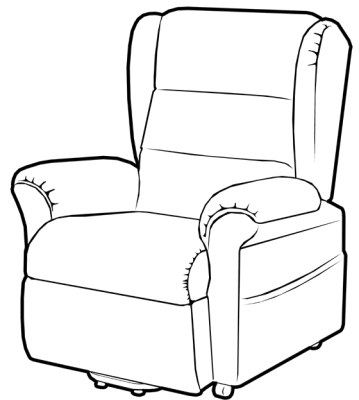

WARRANTY

Your REDGUM Brand:

TARANTO FABRIC 4 Motor Lift Chair (LC0401-2)

is covered by the following warranties from the date of purchase:

- **FRAME (including mechanisms): 5 years**
- **ELECTRONICS: 2 years**
- **FABRICS: 2 years**

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.


If you have any issues or questions about your lift chair please contact your local distributor or REDGUM Brand direct on:

T: +61 8 9248 4180

E: sales@for-de.com.au

A: 1 Business Way, Malaga
Western Australia 6090

**Please keep this User Manual
in a safe place for future reference.**

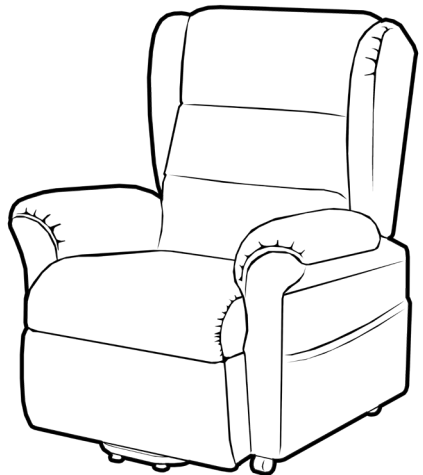

REDGUM BRAND PRODUCT RANGE

REDGUM Brand offer a comprehensive range of products including:

LOUNGE CHAIR PROTECTIVE COVER (LCCOVER)

Ideal way to protect your lift chair from accidental stains

Other relevant products include:

Over bed chair tables

Cushions: Air / Foam

Exercisers

Seat Walkers

Walking Frames

Wheelchairs

Walking Sticks + Quads

Crutches

Reachers

Shower / Bathroom products

Toilet products

Electric Beds

Portable / Fixed Aluminium Ramps

Personal Alarms



REDGUM Brand stock a large range of Spare Parts for all REDGUM brand products.

Please contact REDGUM Brand for further information:

T: +61 8 9248 4180

E: sales@for-de.com.au

**A: 1 Business Way, Malaga
Western Australia 6090**

W: redgumbrand.com.au



DISTRIBUTOR:

REDGUM BRAND
*Daily Living and
Lifestyle Products*



Proudly Australian Owned and Operated

redgumbrand.com.au